

**Voiceover:** Welcome to Prezerve: an easy-to-use symptom-tracking app designed specifically for patients living with graft-versus-host disease, or GVHD

**Onscreen text:**

Prezerve post-transplant symptom tracker

Take Control of GVHD: Track Your Symptoms With the Prezerve App Now

QR Code

**Voiceover:** If you or someone you love has had a bone marrow transplant, Prezerve can help you keep an eye on early and subtle symptoms that may be warning signs of chronic GVHD, or cGVHD.

**Onscreen text:**

If you or someone you love has had a bone marrow transplant, PREZERVE can help you keep an eye on early and subtle signs of cGVHD

**Voiceover:** These changes may not always be obvious — and often, you or your caregiver are the first to notice when something just doesn't feel right.

**Onscreen text:**

- Changes may not always be obvious
- Often, **you or your caregiver are the first to notice** when something doesn't feel right

**Voiceover:** In this video, we'll walk you through how the app works, step by step, so you can feel confident using it to track your symptoms, identify trends over time, and share updates with your care team to ensure you get the treatment you need before your symptoms worsen.

**Onscreen text:**

- Track your symptoms
- Identify trends over time
- Share updates with your care team

**Ensure you get the treatment you need before your symptoms worsen**

**Voiceover:** The Prezerve app makes it easier to share your symptom history with your care team, keeping everyone on the same page and helping make more informed decisions about your treatment. You'll also find helpful resources in the app to support your self-care between appointments.

**Onscreen text:**

- Welcome
- Share symptoms sooner
- Keep everyone informed
- Get selfcare advice

**Voiceover:** Getting started. To get started, search for and download the Prezerve App from the Apple App Store or Google Play.

**Onscreen text:**

- Getting Started
- Sign up
  - o Download

**Voiceover:** When you first launch the app, you'll see a quick reminder of why it's so important to stay on top of your symptoms after a transplant — especially when it comes to catching early signs of cGVHD.

**Onscreen text:**

- Sign up
  - o Getting started

**Voiceover:** Once you've downloaded the app, sign up as a new user with a valid email address — or log in if you already have an account.

**Onscreen Text:**

- Sign up
  - o Enter email
  - o Check email

**Voiceover:** Enter the verification code sent to your email, then follow the prompts to complete your registration and set up your profile.

**Onscreen Text:**

- Sign up
  - o Enter Code
- About You
  - o Enter name
  - o Enter birthday

**Voiceover:** You'll be asked to provide a few details about your transplant, including the type of transplant you received and the date of your procedure, so the app can better tailor your experience.

**Onscreen Text:**

- About you
  - o Enter transplant info

**Voiceover:** Homescreen Overview. Upon registration completion, you'll land on the home screen which will be your starting point for tracking symptoms and exploring helpful resources. From here, you'll be prompted to log any symptoms you may be experiencing. You'll also find articles on health, wellness, and disease management, designed to support you throughout your journey. For those that are interested, the app also includes a daily "Fact or Fiction" trivia question which is a quick way to learn something new each day.

**Onscreen Text:**

- Homescreen Overview

**Voiceover:** Now, let's walk through how to track your symptoms in the app.

**Onscreen Text:**

- Recording symptoms

**Voiceover:** On the home screen, tap the pencil icon in the upper right corner.

**Onscreen text:**

- Click tracker icon
- Select area of symptom

**Voiceover:** You'll see a list of common symptoms to choose from.

**Onscreen text:**

- Select area of symptom

**Voiceover:** If something you're experiencing isn't listed, you can type it in manually.

**Onscreen text:**

- Enter additional symptoms manually

**Voiceover:** Once you select a symptom, you'll be prompted to rate how bothersome it feels using a severity scale. You can update this rating anytime, remove symptoms that have resolved, or log new ones as they appear.

**Onscreen text:**

- Select specific symptom
- Select severity

**Voiceover:** The symptom tracker also lets you mark where on your body the symptom is showing up. Just tap the area on the body outline and follow the prompts.

**Onscreen text:**

- Select location
- Select next

**Voiceover:** Another exciting feature is the ability to add photos. Just tap the camera icon to take a picture or upload one from your photo library. This will serve as a helpful visual for your care team, especially if a symptom is hard to describe.

**Onscreen text:**

- Adding photos
- Tap camera icon
- Choose image source
- Select image

**Voiceover:** You can also rate how much a symptom is affecting your daily life by selecting from a list of impact options or by adding your own notes in your own words if nothing quite fits.

**Onscreen text:**

- Select daily life impact
- Select areas of impact
- Add notes

**Voiceover:** These symptom entries, especially when paired with photos, can help your care team spot early changes in cGVHD and focus treatment where it's needed most.

**Onscreen text:**

- Submit log

**Voiceover:** Resources and Support

**Onscreen text:**

- Resources and Support

**Voiceover:** After you've been tracking your symptoms consistently for some time, the Dr. Summary feature, located in the bottom menu bar, becomes a powerful tool.

**Onscreen text:**

- Select Dr. Summary icon

**Voiceover:** It helps you view your symptom history over time, showing important trends and letting you sort by what's most recent, most bothersome, or most frequently tracked.

**Onscreen text:**

- Select symptom
- View trends

**Voiceover:** You can also generate a PDF report to bring to your next clinic visit or share ahead of time.

**Onscreen text:**

- Send report

**Voiceover:** This report gives your care team a clear snapshot of your experience, with visual insights such as graphs, photo logs, and more. It's a simple but effective way to share if or how your symptoms have changed between visits— helping your providers spot early changes in cGVHD and make more informed decisions about your care.

**Onscreen text:**

- Select mail
- Share
- Tracking Symptoms Helps Your Provider:
  - o Spot early changes in cGVHD
  - o Make more informed decisions about your care

**Voiceover:** Stay Engaged

**Onscreen text:**

- Stay engaged

**Voiceover:** Tap on the Resources and Support tab to explore a library of informative articles and videos, another great feature of the Prezerve app.

**Onscreen text:**

- Explore

**Voiceover:** You can bookmark content you find helpful to access later. You'll also find answers to frequently asked questions and support in the Account menu.

**Onscreen text:**

- Select topics
- View content

**Voiceover:** Keep using Prezerve daily to stay in tune with your body. Small, day-to-day changes should not be overlooked, and consistent tracking helps you and your care team stay ahead of cGVHD.

**Onscreen text:**

- Use Prezerve daily to avoid overlooking subtle changes in your symptoms

**Voiceover:** Don't forget to enable notifications and follow the in-app prompts to make sure you're getting the most out of everything the Prezerve app has to offer — from symptom tracking to wellness content and daily trivia. The Prezerve app puts powerful tools in your hands to help you track symptoms, spot changes early, and share them with your care team to make the most of every visit.

**Onscreen text:**

- Enable notifications
- Follow in-app prompts
- Symptom tracking
- Wellness content
- Daily Trivia
- Share with your care team

**Voiceover:** Whether you're newly post-transplant or managing cGVHD, Prezerve is here to support you, every step of the way.

**Onscreen text:** Whether you're NEWLY POST-TRANSPLANT or CURRENTLY MANAGING cGVHD, Prezerve supports you every step of the way

**Voiceover:** Logging symptoms daily is a simple step to preserve your health

**Onscreen text:**

- Monitor you GVHD at home with the Prezerve App
- Scan the QR code to download the Prezerve App

**Onscreen text:**

- Prezerve
  - o Post-transplant symptom tracker